INSTRUCTION FOR USE

FARBEN COLOURING LIQUID

HS, HT, UHT

SAFETY INSTRUCTIONS

Before use, please carefully read this manual and follow its instructions. Kerox Ltd. should not be liable for any damage resulting from improper use. Please check if the package is not damaged before using the colouring liquid. The colouring liquid can only be used prior to the expiration date.

ATTENTION!

To understand the risks involved in using and handling this product, read and follow the Safety Instructions of this product. The Safety Instruction can be downloaded from the following website: www.kerodental.com. This product can be processed by a dental technician specialist only.

Kerox Farben Colouring liquids are acid based colouring liquids. Observe the safety instructions during use. Residual liquids should be neutralized! They can be neutralized with 0.1 M sodium decarbonate solutions and can later be drained. This process produces carbon dioxide, which in such quantities is not harmful to the human body.

Application of Kerox Farben colouring liquids: for colouring of dental zirconia restorations, the stains completely cover the VITA-Classical colour range. These liquids have been developed for KEROX ZIRCOSTAR zirconia blanks, the correct colouring can only be achieved with the correct matching of ZIRCOSTAR white blanks and Farben colouring liquids!

The required coloring does not change the usability or warranty of the ZIRCOSTAR blanks.

The recommended use:

Type	Farben Ref. Number	Zircostar Ref. Number
HS	ECLxxxHSN	C00N.....
HT	ECLxxxHTN	C00C.....
UHT	ECLxxxUTN	C00U.....

x – VITA shade mark

... – independent characters

PROCESSING INSTRUCTIONS

- **ATTENTION!** Compliance with the rules is essential for the desired shade result.
- Pour the colouring liquid in a plastic container so that it completely covers the crowns that are wished to be coloured.
- Keep the crowns in the colouring liquid for 1 minutes to achieve the desired colour effect.
- After the removal of crowns from the liquid, use a paper towel to blot any excess liquid (from the inside of the crown as well).
- Place the crowns on a tray that does not absorb liquids and is resistant to temperature changes (e.g. sinter tray).
- Place the tray with the crowns on it under an infrared lamp, let it dry for 20-30 minutes, and then put it in a sinter furnace and perform the heat treatment according to the sintering instructions of Kerox.
- Mixing of the colouring liquids is not recommended.
- After use, close the bottle container lid.

SHADES

A1 / A2 / A3 / A3,5 / A4 /
B1 / B2 / B3 / B4 / C1 / C2
C3 / C4 / D2 / D3 / D4

REQUIRED EQUIPMENT

- Metal-free tweezers
- Infrared/drying lamp
- Sinter furnace
- Paper towel

INDIVIDUAL PROTECTIVE EQUIPMENT

- Rubber glove (EN374 series)
- Protective glass (EN 166)
- Protective apron
- Face mask

PROTECTIVE MEASURES

The adequate ventilation of the working area has to be ensured.

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