INSTRUCTION FOR USE

KEROX PREMIA PMMA CAD / CAM DISC

KEROX PREMIA PMMA CAD/CAM DISCS are suitable for fabrication of temporary crowns, bridges and making / designing dental splints. Furthermore, for verifying the bridge on the plaster model/mouth before the final milling process.

KEROX PREMIA PMMA CAD/CAM USAGE, SHADES, MARKING

- We recommend the monocolour version for restorations by VITA shade, pink for base of removable dentures.
A1 / A2 / A3 / B1 / B2 / B3 / BL1 / BL2 / BL3 / PI
- The restorations of **MULTILAYER** are adapted to the natural color gradient of the tooth.
A1M / A2M / A3M / B1M
- The 2-layer (pink and VITA pre-shaded) blanks are recommended for removable dentures.
PIA1 / PIA2 / PIA3 / PIB1.
- The **CLEAR** blanks are recommended for dental splints.
CL / CLO / CLG / CLBE / CLB
(Clear / Orange / Green / Berry / Blue)

CONSIDER WHEN CREATING THE PMMA C&B

- In occlusal region the minimum wall thickness has to be 1.0 mm and 0.6 mm in cervical region.
- The connector profile has to be more than 10 mm² in anterior region and more than 15 mm² in posterior region.
- In posterior region max. two pontics are allowed between two abutment teeth.
- Pay attention to the positioning of the restoration with the multilayer and 2-layer disc. The enamel layer is marked on the top of the disc.

FINISHING C & B AND FIXING IN THE MOUTH

- For grinding use burs for use with acrylic. Polishing is possible with polishing paste and soft hairbrushes. Pay attention not to overheat the material!
- For fixing the bridge use temporary cement.

KEROX PREMIA PMMA CAD/CAM CLEAR type blank is mainly suitable for fabrication of splints and other oral cavity devices. Discs can also be used for fabrication of frameworks for testing within the oral cavity, for working models and as a base for fabrication of moulds in precision casting with metal or press ceramics. Braces are first designed with software for computer-aided design for braces. Thickness and cross-sections that apply to the final material need to be taken into consideration during design of frameworks for precision casting.

TECHNICAL DATA

Flexural strength: >114 MPa
Vickers hardness: >26 HV
E-Module: >2771 MPa
Residual monomer: < 1%

Geometry:

Diameter: 71, 95, 98, 95.5, 100, 115 mm
Thickness: 12, 13, 14, 15, 16, 18, 20, 25, 30 mm

Compatibility with milling machines, diameter:

OS / 98 mm / Wieland®
ZZ / 95 mm / ZirkonZahn®
AG / 71 mm / Amann Girrbach®
MIK / 115 mm / unique demand

CONTRA-INDICATION

In case of known allergies or hypersensitivity to acrylic materials do not use the PMMA product.

WARNING

Wear a dust mask during fabrication.
Disinfect PMMA before fixing it in the mouth
After regular controls, the dentist can determine how long the product can stay in the mouth.
After removal from the mouth, the product must be considered as hazardous waste.

In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (info@keroxdental.net) and the competent authority of the Member State, where the user is established.

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