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KEROX FARBEN és KEROX EFFEKTEN

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INSTRUCTIONS FOR USE

KEROX FARBEN and KEROX EFFEKTEN

COLOURING LIQUID FOR STAINING OF DENTAL ZIRCONIA CERAMICS

TABLE OF CONTENTS

1. PRODUCT DESCRIPTION
2. USERS
3. INFORMATION FOR SAFETY
4. SAFE DISPOSAL INFORMATION
5. INTENDED USE
6. INDICATIONS, CLINICAL BENEFITS
7. CONTRAINDICATIONS, RESIDUAL RISKS
8. TRANSPORT AND STORAGE
9. PROCESSING INSTRUCTIONS FOR DENTAL TECHNICIANS
10. COLOURING
- 10.1 DIPPING METHOD
- 10.2 BRUSHING METHOD
11. SINTERING
12. POST-TREATMENT
13. INSERTION OF THE PREPARED RESTORATIONS
14. SHELF LIFE
15. SYMBOLS-MARKING

Instructions for use is applicable to items with the following CATALOGUE NUMBERS **REF.**:

ECLxxxHSN	ECLxxxHTN	ECLxxxUTN
-----------	-----------	-----------

Which are indicated the following product types: **FARBEN** coloring liquid for ZIRCOSTAR® High Strength (HS), **FARBEN** coloring liquid for ZIRCOSTAR® High Translucent (HT) and **FARBEN** coloring liquid for ZIRCOSTAR® Ultra High Translucent (UHT) zirconia materials.

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Which are indicated the following product types: **EFFEKTEN Grey, Braun, Pink Lite** and **Pink Dark** colouring liquids for ZIRCOSTAR® zirconium dioxide materials.

1. PRODUCT DESCRIPTION

You have chosen **KEROX Farben** or **KEROX Effekten** coloring liquids. The products are suitable for the surface colouring and shading of biocompatible, fixed zirconia dental restorations (crowns, bridges, superstructures, so-called inlays and onlays, etc.) designed and produced by a technician in a dental laboratory based on the dentist anamnesis to achieve lifelike aesthetics as the natural tooth colour. Restorations prepared by the application of **Farben** colouring liquids perfectly meet the aesthetic, functional and health requirements to natural teeth, which are fundamental to maintain the patient's health conditions. To mimicking the characteristics of the patient's unique tooth shades **Effekten** product variations are available to reproduce the lifelike cervical discolorations, interdental spaces and shades and translucency of tooth edges.

The product group is classified as Class IIa according to its intended use and risk classification in accordance with Chapter V, Section 1, Article 51 of Regulation (EU) 2017/745.

2. USERS

The processing of ZIRCOSTAR® pre-sintered zirconium dioxide (zirconia) product treated with **Farben** or **Effekten** colouring liquids can only be performed by a qualified DENTAL TECHNICIAN. The insertion of the prepared restoration into the PATIENTS' oral cavity can only be completed by DENTIST under the appropriate conditions. The above listed product is only intended for professional use for the mentioned users.

3. INFORMATION FOR SAFETY

Please read and completely follow all the information in this instruction for use before use! KEROX Ltd. does not take responsibility for results and events derived from misuse due to deviation of the described instructions. According to the intended use of the product it shall be processed in a suitable dental laboratory complies with safety standards and provided adequate processing equipment. During the product processing at dental laboratory observe work safety and personal protection. The colouring liquids have acidic pH-value and in contact with skin it has irritating effect. Always wear protective gloves. In case of contact with skin water flush promptly.

In contact with eyes by accident, use a neutralizing eyewash immediately! The use of safety glasses is recommended!

ATTENTION, not to get liquid on clothes or surfaces it will result contamination. The liquids are CORROSIVE in contact with metals, therefore avoid using tools or equipment containing metal (tweezers, brushes, jars).

Prior to use the product, read information described in the SAFETY DATA SHEET (SDS) and follow the instructions.

Please check the following before use of the product:

- Damage-free packaging, unopened bottle.
- Condition of the item: The liquid in the bottle is transparent, turbidity-free and sediment-free,
- whether the product contains the trademark of KEROX Ltd. and the basic product information and identifiers (GTIN, REF, SN, etc.)

In case of any observed deviations from the described, please don't start use the product! If you experience degradation or damage of the properties or performance of the device-including errors derived from misuse due to ergonomic characteristics- as well unwanted side effects. In this instance contact the manufacturer immediately on the provided contacts.

Please contact us immediately in case of any incidents or serious adverse events related to the device and/or the competent authority. Contact information of the Hungarian authorities:

NINGYK-National Public Health and Pharmaceutical Center
H-1135 Budapest, Szabolcs street 33.

Tel.: +36 1 886 9300

E-mail: amd.vig@ningyk.gov.hu

SAFE DISPOSAL

The product shall be neutralized with SODIUM HYDROGEN CARBONATE (CAS number 144-55-8). The neutralized solution suitable to pour into the sewage network.

The user of the device shall handle the product package in accordance with Directive 94/62/EC or national and local regulations.

4. INTENDED USE

In case of the adequate processing method described in this IFU's PROCESSING GUIDE dental restorations prepared from the mentioned ZIRCOSTAR® product types treated with **Farben** or **Effekten** colouring liquids comply with Class 5 of the ISO 6872:2024 (E) standard in terms of their mechanical and chemical properties. The application of the colouring liquids provides the optimal results solely together with non-pre-shaded ZIRCOSTAR® materials. The **Farben** and **Effekten** products are suitable for either the brushing or dipping colouring techniques. Using **Farben** product variations it is possible to prepare dental restorations identical to 16 parts of the *VITA classical A1 -D4* shade guide.

*VITA classical A1-D4®- registered trademark, VITA Zahnfabrik H. Rauter GmbH & Co. KG in Bad Säckingen

5. INDICATIONS

KEROX ZIRCOSTAR® pre-sintered zirconia ceramics treated with **Farben** or **Effekten** colouring liquids are suitable for the preparation of fixed dental restorations which decrease or abolish patients' deterioration of health resulted from tooth loss or damage. The prepared restorations are applicable to eliminate extended tooth loss either. By the application of the prepared ceramic restorations further deterioration of the patient's health can be stopped moreover lifelike dentures can be made similar aesthetics to natural hydroxyapatite-based teeth. It is especially beneficial for patients with sensitivity to metals.

The applicability shall determine by qualified dentist only!

6. CONTRAINDICATIONS

Restoration prepared by using KEROX **Farben** or **Effekten** colouring liquids may result unwanted effects in case of patients struggling with HYPERSENSITIVITY.

8. TRANSPORT AND STORAGE

KEROX **Farben** and KEROX **Effekten** colouring liquids are packed and distributed in transparent HDPE bottles. Please always pay attention to the followings in case of transport, storage, and use of the products:

- The device shall be transported and stored in its original package. Pay attention to the condition of the product labels on the items! Try to preserve its content and condition during the processing.
- Close the bottle immediately as soon as possible after use. Always keep closed when not in use.
- Keep the indicated storage conditions! The device shall be stored between 5°C and 40°C. Keep the product away from radiant heat and sunlight.
- In case of the working process is carried out near an infrared lamp or other light sources that emit radiant heat, keep the bottle of coloring liquid away from them.
- DO NOT pour the liquid back in its bottle!
- The material shall not contact with liquids, other contamination resulting substances especially metal or mineral particulates during storage conditions.
- Do not mix it with other coloring materials, do not dissolve substances in it!

9. PROCESSING GUIDE FOR DENTAL TECHNICIANS

Please keep the transport and storage terms pay attention to the following:

- Wear protective gloves, safety glasses (EN 166) during the processing.

Follow the general working environment safety protocol and accident prevention regulations.

Ensure proper ventilation in the work area, use fume hood if possible.

10. COLOURING

1. Before the application of colouring liquids milled zirconia workpiece surface shall be completely free from remnant particles and grease.
2. In case of wet machining method, the workpiece shall be dried before applying the coloring liquid.
3. Handle the workpieces with metal-free tweezers. USE CLEAN protective gloves, since grease and other contaminants from the skin surface can negatively affect the absorption of the liquid on the ceramic surface.
4. Use the recommended type of **Farben** or **Effekten** variations with the chosen ZIRCOSTAR® zirconia product type according to the desired tooth shades and aesthetics. Always pour out only the adequate amount of liquid according to the applied colouring method. Always close the bottles.
5. The adequate amount of liquid shall be poured out into a suitable container (plastic or glass). Consider the size of the workpiece to be coloured when apply the dipping method and choose the appropriate beaker volume and liquid level.

10.1 IN CASE OF DIPPING METHOD the ceramic workpiece shall be immersed evenly into the coloring liquids for at least **30 seconds**. ATTENTION! If the prepared restorations (bridges) intermediate unit's volume is higher, it absorbs more coloring liquid which results more intense shade. To avoid the increased shade intensity in case of the intermediate bridge units wet with a brush the milled restoration's surface by using distilled or ion exchange water before dipping.

6. As the dipping was finished, place the workpiece on a paper napkin which doesn't result contamination. Let it drain the remnant moisture from the surface.

10.2 IN CASE OF BRUSHING METHOD use more metal-free brushes for work.

If switching the colouring type or shade, always wash and dry the brush before using it again. This step helps avoiding cross-contamination and unwanted dilution of the liquids. Use different brushes for **Effekten** liquids.

7. As the brushing was finished, place the workpiece on a paper napkin which doesn't result contamination. Let it drain the remnant moisture from the surface.

8. ALWAYS dry the workpiece after the colouring process. The drying process shall be carried out under infrared lamp (250W) or special drying ovens. The set temperature should not exceed 100° C! The drying interval depends on the workpiece size and geometry. On average, it takes 20-30 minutes.

9. Use metal-free heat-resistant trays that won't damage or react with the material of the zirconia (aluminum oxide or silicate fiber materials).

11. SINTERING

1. Prepare the sintering crucible for sintering, fill it with zirconium dioxide sintering beads.
2. The placement of the milled workpieces in the sintering crucible shall be optimized to reach the best heat transfer during the process. If there is a need to put more workpieces into the sintering crucibles use suitable ceramic spacers. The milled workpiece should not be contacted with other ceramic pieces or the material of the sintering crucibles. In case of the application of faster sintering program never place more sintering crucibles into the furnace or put more level of workpieces into the crucible.
3. In case of sintering process of long-span bridges the workpiece shall be placed vertically into the furnace together with its support and brace elements.
4. Set the program of the sintering furnace according to the selected ZIRCOSTAR® instructions for use with the corresponding **REF.** type of the KEROX **Farben** or **Effekten** coloured zirconia.
5. Start the sintering program after the insertion of workpieces the sintering crucibles.
6. **ATTENTION!** Follow the dental furnace manufacturer's handling and safety instructions! Remove the workpiece from the oven only after it has completely cooled down.
7. Inspect the condition of the sintered workpiece(s). Separate the sintered workpiece by the application of water- or air-cooled cutting tool from the support frame. The separating process avoid overheat effect and effects which could awake mechanical stresses in the material result cracks. Next clean, degrease and dry the workpiece!

12. POST-TREATMENT

Sintered KEROX ZIRCOSTAR® zirconium dioxide material coloured with **Farben** or **Effekten** is suitable for further polishing and the applicability of surface glaze material layers. In case of polishing method, the applicability of the glaze material may decrease. Do not polish the zirconia surface when working with porcelain veneered structures.

13. INSERTION OF THE PREPARED RESTORATIONS

Zirconia-compatible adhesives shall be used in case of insertion of dental restorations prepared from KEROX ZIRCOSTAR® material coloured with **Farben** and **Effekten** to the oral cavity. Follow the adhesive manufacturer's instructions during handling.

13. SELF LIFE

The product can be used without any decrease in its quality in unopened bottle, stored in a place protected from light and heat FOR 1 YEAR FROM THE MANUFACTURING DATE.

After it was opened it shall be stored in the specified storage and cleanliness conditions without any deterioration.

14. SYMBOLS AND MARKING

	MANUFACTURER		DON'T USE IF THE PACKAGE IS DAMAGED!
	USE BEFORE		STORE DRY
	FRAGILE, HANDLE WITH CARE!		FOLLOW THE INSTRUCTIONS!
	STORE TEMPERATURE		UNIQUE DEVICE IDENTIFIER
	CATALOGUE NUMBER		MEDICAL DEVICE

Manufacturer: KEROX Ipari és Kereskedelmi Kft.
Address: H-2038 Sósokút, Keroxu. 1., Hungary
Telephone: +36 23 3560700
E-mail: info@kerodental.net
Web: <https://kerodental.com>