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KEROX ZIRCOSTAR® PRE-SINTERED ZIRCONIUM DIOXIDE (ZIRCONIA) CERAMIC

TABLE OF CONTENTS

1. PRODUCT DESCRIPTION
2. USERS
3. INFORMATION FOR SAFETY
4. SAFE DISPOSAL INFORMATION
5. INTENDED USE
6. INDICATIONS, CLINICAL BENEFITS
7. CONTRAINDICATIONS, RESIDUAL RISKS
8. TRANSPORT AND STORAGE
9. PROCESSING INSTRUCTIONS FOR DENTAL TECHNICIANS
10. DESIGNING CONSIDERATIONS (CAD-CAM)
11. MILLING AND MACHINING CONSIDERATIONS
12. COLOURING AND SINTERING
13. POST-TREATMENT
14. USE OF FINISHED RESTORATIONS
15. ZIRCOSTAR® MATERIALS COMPOSITION
16. ZIRCOSTAR® PROPERTIES OF SINTERED MATERIAL
17. SYMBOLS-MARKING

Instructions for use is applicable to items with the following CATALOGUE NUMBERS:

B-UDI	PRODUCT	CATALOGUE NUMBER REF
5996612CS00002J	HS	CYYNXXXXXX
5996612CTS000022	HT	CYXCXXXXXX
5996612CUS00002F	UHT	CYUXXXXXXX

which is intended to mark the following types:

ZIRCOSTAR® High Strength (HS), ZIRCOSTAR® High Translucent (HT) and ZIRCOSTAR® Ultra High Translucent (UHT) zirconia blanks.

1. PRODUCT DESCRIPTION

You have chosen KEROX ZIRCOSTAR®. The product is suitable for the preparation of biocompatible fixed dental zirconia restorations such as crowns, bridges, chewing surface inlays and onlays designed according to the dentist anamnesis via CAD/CAM and manual processing by professional dental technicians.

Dental restorations prepared from ZIRCOSTAR® perform excellent lifelike aesthetics mimicking natural teeth. Restorations fulfill functional and medical requirements, which are fundamental element of the patient's health preservation. The following restoration types prepared from ZIRCOSTAR® zirconia ceramics are not permitted to use:

DENTAL IMPLANTS AND ABUTMENTS- AS A CONNECTED ELEMENT TO IMPLANTS

The product group is classified as Class IIa according to its intended use and risk classification in accordance with Chapter V, Section 1, Article 51 of Regulation (EU) 2017/745.

2. USERS

The processing of ZIRCOSTAR® pre-sintered zirconium dioxide (zirconia) product can only be performed by a qualified DENTAL TECHNICIAN. The insertion of the prepared restoration into the PATIENTS' oral cavity can only be completed by DENTIST under the appropriate conditions. The above listed product is only intended for professional use for the mentioned users.

3. INFORMATION FOR SAFETY

Please read and completely follow all the information in this instruction for use before use! KEROX Ltd. does not take responsibility for results and events derived from misuse due to deviation of the described instructions. According to the intended use of the product it shall be processed in a suitable dental laboratory complies with safety standards and provided adequate processing equipment. During the product processing at dental laboratory observe work safety and personal protection. The machined ceramic has sharp surfaces or edges that could cause injury. Read and follow the materials SAFETY DATASHEET information before use.

Please check the following before use of the product:

- Damage-free packaging
- Condition of the item (free from cracks discoloration and contaminations) whether the product contains the trademark of KEROX Ltd. and the basic product information and identifiers (GTIN, REF, SN, etc.)

In case of any observed deviations from the described, please don't start use the product! If you experience degradation or damage of the properties or performance of the device-including errors derived from misuse due to ergonomic characteristics- as well unwanted side effects.

Table 1: RECOMMENDATIONS OF INTENDED USE

REF	TYPE	RECOMMENDATIONS	Anatomical ANTERIOR CROWN	ANTERIOR COPING	Anatomical ANTERIOR BRIDGE	ANTERIOR BRIDGE COPING	VENEER	INLAY/ ONLAY	Anatomical POSTERIOR CROWN	POSTERIOR COPING	Anatomical POSTERIOR BRIDGE	POSTERIOR BRIDGE COPING	LONG SPAN BRIDGE	LONG SPAN coping	
CYYNXXXXXX	High Strength (HS)	APPLICABILITY	—	***	—	***	—	—	—	***	—	***	—	***	
		UNITS (pcs)	—	1	—	>2	—	—	—	—	0.7	—	>2	—	14
		Recommended minimal wall thickness (mm)	—	0.5	—	0.5	—	—	—	—	—	—	0.5-0.7	—	0.5-0.7
		Minimal cross-section of CONNECTORS (mm ²)	—	—	—	7	—	—	—	—	—	—	9	—	7-9
CYXCXXXXXX	High Translucent (HT)	APPLICABILITY	*	***	*	—	*	***	***	***	***	—	*	***	
		UNITS (pcs)	1	1	>2	—	—	—	—	—	—	>2	—	14	14
		Recommended minimal wall thickness (mm)	0.5	0.5	0.5	—	0.5	0.5	—	0.5-0.7	0.5-0.7	—	—	0.5-0.7	0.5-0.7
		Minimal cross-section of CONNECTORS (mm ²)	—	—	7	—	—	—	—	—	—	—	9	—	7-9
CYUXXXXXX	Ultra High Translucent (UHT)	APPLICABILITY	***	***	***	—	***	*	***	—	***	—	***	—	
		UNITS (pcs)	1	1	>2	—	—	—	—	—	—	>2	—	14	—
		Recommended minimal wall thickness (mm)	0.6	0.6	0.6	—	0.6	0.6	—	0.6-0.8	—	0.6-0.8	—	0.6-0.8	—
		Minimal cross-section of CONNECTORS (mm ²)	—	—	7	—	—	—	—	—	—	—	7-12	—	7-9

It is recommended to keep 1 mm safety distance in case of the design from the edges of the zirconia blank.

Legend: — not applicable; * suitable; *** recommended

In this instance contact the manufacturer immediately on the provided contacts.

Please contact us immediately in case of any incidents or serious adverse events related to the device and/or the competent authority. Contact information of the Hungarian authorities:

NNGYK-National Public Health and Pharmaceutical Center
H-1135 Budapest, Szabolcs street 33.
Tel.: +36 1 886 9300
E-mail: amd.vig@nngyk.gov.hu

4. SAFE DISPOSAL

Following the processing by technicians and application by the dentist at the end of the life cycle of the product shall be handled as hazardous waste material. In accordance with Directive 2008/98/EC or national and local regulations. The user of the device shall handle the product package in accordance with Directive 94/62/EC or national and local regulations.

5. INTENDED USE

In case of the adequate processing method described in this IFU's PROCESSING GUIDE dental restorations prepared from the mentioned ZIRCOSTAR® product types comply with Class 5 of the ISO 6872:2024 (E) standard in terms of their mechanical and chemical properties. See the intended and safety usability and performance of the prepared restorations listed in Table 1. The product can be kept and used for 10 years, subject to compliance with the transport and storage conditions.

6. INDICATIONS, CLINICAL BENEFITS

KEROX ZIRCOSTAR® pre-sintered zirconia ceramics are suitable for the preparation of fixed dental restorations which decrease or abolish patients' deterioration of health resulted from tooth loss or damage. The zirconia material is durable, and its resistance to chewing force is effective. The dental restorations made of the material are also suitable for eradicating a large number of missing teeth, or a replacement can be made with minimal tooth material removal. The prepared restorations are applicable to eliminate extended tooth loss either. Due to material properties of ZIRCOSTAR® ceramic dental restorations could be prepared with excellent (0.01 mm) fit. By the application of the prepared ceramic restorations further deterioration of the patient's health can be stopped moreover, due to the optical properties of ZIRCOSTAR® lifelike dentures can be made similar aesthetics to natural hydroxyapatite-based teeth. It is especially beneficial for patients who are sensitive to metals. A dental prosthesis made by using zirconia ceramics enables the examination of the patient with imaging diagnostic methods (MRI). The applicability and the intended patient population are determined exclusively by qualified dentist.

7. CONTRAINDICATIONS, RESIDUAL RISKS

CONTRAINDICATIONS:

The use of ZIRCOSTAR® restorations is not recommended in cases of temporomandibular joint disorders, bruxism, or any clinical situation in which the treating dentist considers the material unsuitable. Rarely, hypersensitivity may occur.

RESIDUAL RISKS:

Despite risk-reduction measures, the following residual risks may remain:

- fracture or chipping under excessive occlusal or lateral loading,
 - veneering ceramic chipping in cases of non-optimal occlusion,
 - misfit or marginal discrepancy causing irritation,
 - colour or translucency deviation compared to adjacent dentition,
 - material weakening due to processing errors (grinding, cooling, sintering),
 - de-bonding resulting from cementation or adhesion issues.
- The clinical relevance of these residual risks depends on the individual case; the treating dentist provides patient-specific information accordingly.

8. STORAGE AND TRANSPORT

KEROX ZIRCOSTAR® zirconia ceramic is delivered in a pre-sintered state, ready for machining. Please observe the following guidelines during transportation, storage, and use of the product:

- Always store the product in its original packaging! Pay attention to the condition of the label found on the side of the product. Take care to preserve its integrity throughout use.
 - Always comply with the specified storage conditions!
- Keep the product in a dry location with low humidity, and within an ambient temperature range of -20°C to +50°C.
- Do not expose the pre-sintered zirconium dioxide block to excessive mechanical stress (e.g. impact, abrasion, etc.).

- Ensure that the product does not come into contact with liquids, or other contaminating substances, particularly coarse or fine metallic dusts.
- During processing, always wear protective gloves, safety goggles, and a dust mask rated FFP3 according to EN 149! Always follow general occupational safety regulations.

Commence use of the product only in a properly equipped, controlled, and safety-compliant working environment.

9. PROCESSING INSTRUCTIONS FOR DENTAL TECHNICIANS

Please ensure that all requirements regarding the storage and transportation of the product are strictly observed throughout its use. In particular, pay attention to the following: Comply with all general occupational safety and accident prevention regulations. Begin use of the product only in an appropriately equipped and safe working environment. The selection of tools and accessories used for processing must be made by a qualified dental technician.

10. DESIGN CONSIDERATIONS (CAD-CAM)

1. Ensure that the designed restoration's wall thicknesses are adequate and maximize the number of cross-sectional connectors. During the designing method of the zirconia restorations consider the recommended usage in Table 1.

2. Observe and examine the to be designed workpiece, position it, run a simulation of the planned milling strategy regarding to the designed restoration's characteristics and properties. Carry out calculations with shrinkage factor (magnification level) marked on the product labels. In case of the design of the workpiece's braces and connectors estimate the effects of awaken mechanical forces and try to reduce it. Ensure uniform heat transfer during the sintering process by decrease difference in the materials thickness in the workpiece.

3. Insert the ceramic blank into the milling machine as described in the CAD-CAM system's manufacturers manual.

11. MILLING, MACHINING CONSIDERATIONS

4. Start the milling process. ATTENTION! Always follow the milling system manufacturer's instructions in case of handling and safety protocol.

5. By finishing the processing removes the ceramic workpiece from the CAM milling system and inspect it visually to avoid damages. In case of any observed damage do not continue the machining.

6. To separate the milled workpiece from the zirconia blank and in case of the following cutting steps use suitable diamond-coated milling and cutting tools.

7. Remove the remnant dust and particles carefully after the milling from the surface of the workpiece by oil-free compressed air. Never carry out sintering process with uncleaned workpieces. In case of wet machining method, the workpiece shall be dried before sintering

12. COLOURING AND SINTERING

8. For final shade adjustment, only dental zirconia colouring liquids may be used. Compatibility of KEROX FARBEN® and KEROX EFFEKTEN® has been verified. The use of other systems falls within the professional competence of the dental technician.

9. If colouring techniques were applied on the milled restoration ALWAYS dry the workpiece! The drying process shall be carried out under infrared lamp (250W) or special drying ovens. The set temperature should not exceed 100°C! The drying interval depends on the workpiece size and geometry.

10. Prepare the sintering crucible for sintering, fill it with zirconium dioxide sintering beads.

11. The placement of the milled workpieces in the sintering crucible shall be optimized to reach the best heat transfer during the process. If there is a need to put more workpieces into the sintering crucibles use suitable ceramic spacers. The milled workpiece should not be contacted with other ceramic pieces or the material of the sintering crucibles. In case of the application of faster sintering program never place more sintering crucibles into the furnace or put more level of workpieces into the crucible.

12. In case of sintering process of long-span bridges the workpiece shall be placed vertically into the furnace together with its support and brace elements.

13. Working with the indicated KEROX ZIRCOSTAR® REF in this instruction always follow the attached sintering programs (see Table 2).

14. Start the sintering program after the insertion of workpieces the sintering crucibles. ATTENTION! Follow the dental furnace manufacturer's handling and safety instructions!

15. Remove the workpiece from the oven only after it has completely cooled down.

16. Inspect the condition of the sintered workpiece(s). Separate the sintered workpiece by the application of water- or air-cooled cutting tool from the support frame. During the separating process avoid overheat effect and effects which could awake mechanical stresses in the material result cracks. Next clean, degrease and dry the workpiece! In the case of anatomical restorations, carefully polish the occlusal surfaces in order to avoid wear on the surface of the antagonist natural teeth.

13. POST-TREATMENT

Sintered KEROX ZIRCOSTAR® zirconium dioxide material is suitable for further polishing and the applicability of surface glaze material layers. In case of polishing method, the applicability of the glaze material may decrease. Do not polish the zirconia surface when working with porcelain veneered structures.

14. INSERTION OF THE PREPARED RESTORATIONS

Zirconia-compatible adhesives shall be used in case of insertion of dental restorations prepared from KEROX ZIRCOSTAR® material to the oral cavity. Follow the adhesive manufacturer's instructions during handling. The denture must be cleaned and checked in accordance with the DENTIST recommendation. The expected lifetime of the replacement is 10-15 years with proper cleaning and inspection.

IF THE PRODUCT WAS SUFFERED DEFORMATION, DAMAGED OR CONTAMINATED DURING PROCESSING OR FITTING INTO THE ORAL CAVITY BY THE DENTIST, IT IS NOT PERMITTED TO BE USED!

15. ZIRCOSTAR® MATERIAL'S COMPOSITION

Oxides	Percentage by weight
ZrO ₂	90.2-94.3 wt%
Y ₂ O ₃	5.7-9.8 wt%
Al ₂ O ₃	<0.25 wt%
SiO ₂	<0.02 wt%
Fe ₂ O ₃	<0.02 wt%
Na ₂ O	<0.02 wt%

16. PROPERTIES OF SINTERED ZIRCOSTAR®

ZIRCOSTAR® HIGH STRENGTH	
Radioactivity	<15 Bq/kg
Flexural strength	1443 MPa
Fracture Toughness	10,72 MPa·m ^{1/2}
CTE	10,3 × 10 ⁻⁶ 1/°C
Chemical solubility	5,4 µg/cm ²
Sintered density	6,05 g/cm ³
ZIRCOSTAR® HIGH TRANSLUCENT	
Radioactivity	<13 Bq/kg
Flexural strength	1350 MPa
Fracture Toughness	12,27 MPa·m ^{1/2}
CTE	9,9 × 10 ⁻⁶ 1/°C
Chemical solubility	16,1 µg/cm ²
Sintered density	6,05 g/cm ³
ZIRCOSTAR® ULTRA HIGH TRANSLUCENT	
Radioactivity	<11 Bq/kg
Flexural strength	1175 MPa
Fracture Toughness	5,0 MPa·m ^{1/2}
CTE	9,6 × 10 ⁻⁶ 1/°C
Chemical solubility	1,8 µg/cm ²
Sintered density	6,05 g/cm ³


17. SYMBOLS-MARKING

	MANUFACTURER		DON'T USE IF THE PACKAGE IS DAMAGED!
	USE BEFORE		STORE DRY
	FRAGILE, HANDLE WITH CARE!		FOLLOW THE INSTRUCTIONS!
	STORE TEMPERATURE		UNIQUE DEVICE IDENTIFIER
	CATALOGUE NUMBER		MEDICAL DEVICE

Manufacturer: KEROX Ipari és Kereskedelmi Kft.
Address: Kerox utca 1., Sósút-2038, Hungary
Telephone: +36 23 560 700
E-mail: info@keroxdental.net
Web: https://keroxdental.com

Table 2: SINTERING PROGRAM OF DENTAL FURNACE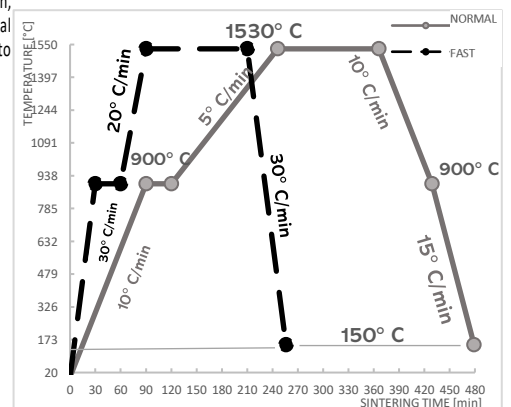

SINTERING PROGRAM	NORMAL	FAST
APPLICABILITY	ALL TYPES OF RESTORATIONS	IN CASE OF MAX. 4-UNIT RESTORATIONS
STEP 1	↗ 10°C/min → 900°C	↗ 30°C/min → 900°C
STEP 2	→ 900°C 30 min	→ 900°C 30 min
STEP 3	↗ 5°C/min → 1530°C	↗ 20°C/min → 1530°C
STEP 4	→ 1530°C 120 min	→ 1530°C 120 min
STEP 5	↘ 10°C/min → 900°C	↘ 30°C/min → 900°C
STEP 6	↘ 15°C/min → *150°C	↘ 35°C/min → *150°C

* or passive cooling